



Storage

Keep container tightly closed. Keep away from heat, light and moisture. Store in a cool place, but not below -5°C for a longer period of time.

Physical properties

Appearance: clear, colourless fluid of low viscosity
Density: $D = 0.866 - 0.884 \text{ g/cm}^3 (20^\circ\text{C})$
Flash point: 19°C according to DIN 51755
Temperature resistance: between 0 and +40°C.

Toxicology

The hand and skin disinfectant C 20 contains as effective and further components only toxicologically known substances which are considered to be safe. According to the current level of knowledge, all raw materials of the medical device have neither mutagenic nor embryotoxic nor teratogenic nor carcinogenic properties. If the product gets into contact with mucous membranes or injured skin, a slight irritation may arise, the skin may slightly redden or burn. For this reason, do not use C 20 for disinfecting mucous membranes or injured skin.

Ecology

C 20 solutions are ecologically safe. If C 20 gets into the waste water, all ingredients are biodegradable. The packing is made of polyethylene and can therefore be exploited both materially and thermally. To recycle, rinse containers and bottles with water.

Caution

Highly flammable. Flash point in accordance with DIN 51755: 19°C. Keep away from open flames and sources of ignition. If the solution is spilled, immediately take appropriate measures against fire and explosion. Appropriate measures are for example the absorption of the spilled liquid and the dilution with water, ventilation of the room and the removal of sources of ignition. For external use only. Avoid contact with the eyes.

The information on our products is based on detailed internal and external scientific examinations, and recommendations are given to the best of our knowledge. Liability is being judged in accordance with our General Terms of Sale and Delivery which are available at any time upon request.

C 20 - propan-2-ol 70% (v/v) - **Composition:** Active ingredients: 100 g solution contains 62.8 g of propan-2-ol. Other ingredients: purified water. **Areas of application:** Hygienic and Surgical hand disinfection, skin disinfection before simple injections and punctures of peripheral vessels, skin disinfection before operations and punctures of joints, disinfection of skin which is rich in sebaceous glands, cold compresses. **Contraindications:** C 20 is not suitable for disinfecting injured skin. Do not use C 20 if you are allergic to one of the ingredients. **Side effects:** The skin may redden or burn when rubbing in C 20. **Notes:** Keep container tightly closed. Pharmaceutical product, keep out of reach of children. Should not be used after the expiry date has lapsed. **Pharmaceutical entrepreneur and manufacturer:** orochemie, D-70806 Kornwestheim.

Complementary products from oro® Hygienesystem

Instrument disinfection • Disinfection of surfaces and inventory • Disinfection and care of hands and skin • Disinfection of special applications • Accessories

Information as of 01/10